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EN	Dental dam
BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY	
TO BE USED BY DENTIST ONLY	

INTENDED USE

RUBBER-DAM device is intended for use during dental procedures to isolate the treatment area and protect the gums.

INSTRUCTIONS FOR USE

- Place dental dam on template then adjust.
- Use a marker to mark a point that defines a specific tooth to be isolated.
- Use a punch to cut the hole.
- Place the cofferdam on the metal frame.
- Place the clamp wings in the hole.
- Using a rubber dam forceps open the clamp and place it on the tooth.

CAUTION

This product contains natural rubber latex which may cause allergic reactions.

PRECAUTIONS

It is intended for continuous use for less than 60 minutes during dental procedures.

Any serious incident involving the product should be reported to the manufacturer and the competent authority of the State in which the user or patient is domiciled.

Do not use the products if you found any holes or tearing.

STORAGE

Keep dry.

Store at a temperature between 5°C (41°F) and 40°C (104°F).

PACKAGING

The package contains 36 pieces.

VARIANTS AVAILABLE

RUBBER-DAM BLUE PLAIN MEDIUM, Dental dam; medium 0,18 mm, odourless, blue
RUBBER-DAM GREEN MINT MEDIUM, Dental dam; medium 0,18 mm, mint, green
RUBBER-DAM BLACK MINT MEDIUM, Dental dam; medium 0,20 mm, mint, black
RUBBER-DAM BLUE PLAIN HEAVY, Dental dam; thick 0,22 mm, odourless, blue
RUBBER-DAM GREEN MINT HEAVY, Dental dam; thick 0,22 mm, mint, green

AFTER USE, HAND OVER FOR DISPOSAL TO AN AUTHORIZED RECIPIENT. REUSE MAY CAUSE INFECTION.

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